carlett.

ENJOY THE ECOMOTION

ES

CA

EN

FR

DE

IT

Comfort

ASSIST

español

Instrucciones

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english

Instructions

Thank you for purchasing this Carlett product. Remember to keep the receipt, because it is your only proof of guarantee if the product is defective.

Any defect arising from misuse of the product by the user and the possible normal deterioration of certain components (wheels and textile components) due to wear and tear of the product are excluded from the guarantee.

Before using the trolley, please read the instructions for its proper use carefully.

We hope that you enjoy this product, which has been designed to help improve its users' quality of life.

Thank you!

Assist, shopping and sitting comfortably

The Assist trolley, allows the user to go shopping and walk with the peace of mind of being able to sit down and rest whenever necessary.

EN

It has a safety or parking brake, which is essential for using the trolley as a seat. To activate the safety brake, pull the brake lever handle backwards. To take the brake off and start moving again, push the lever forwards (see image no. 1).

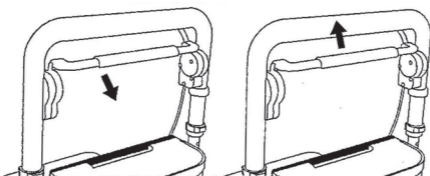
Image 1. Safety brake

It incorporates an instant braking system which is activated by pulling it with your hands until the trolley stops. To disable it and/or continue walking, simply stop pulling the brake lever (see image no. 2).

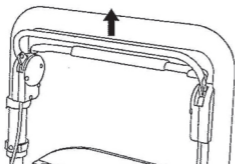
Image 2. Instant brake

The product also has a system for adjusting the degree of rotation of the two front swivel wheels according to your

needs. The wheels can be completely fixed, with a 90° rotation or fully rotating (360°). To change the degree of rotation, press the adjustment mechanism located in the front wheel and adjust it as required (see image no. 3).

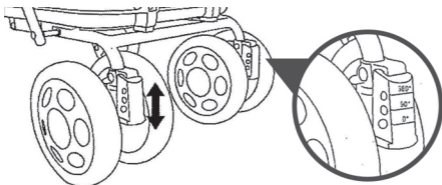
Image 3. Rotating wheels

It has a height adjustment mechanism for the handlebar with three positions. To change the height, remove the blockers from the holes located on the left and right of the handlebar, move it to the desired height and secure it again by introducing the blockers into the correct holes on both sides (see image no. 4).

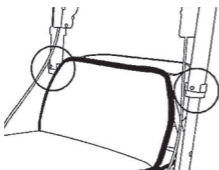
Image 4.
Adjustable handlebar

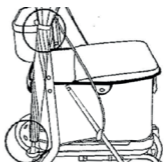
Image 5. Umbrella suport

EN

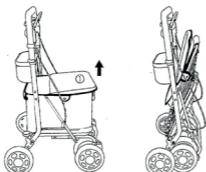
Image 6. Folding system

It contains a rack for umbrellas, which can be fastened using the strap (see image no. 5).

To fold up and store the product, you only need to pull the seat cushion upwards (see image no. 6).

It has a removable inner bag with handles which is easy to clean.

Material

- Chassis: aluminium
- Textile components: Polyester

- 
- 
- Wheels: EVA material (this material means that the wheels are silent, even on rough surfaces).
 - Tyres: polypropylene

Specifications

- Unfolded product: L65 x W52.5 x H88.5 - 98.5 cm
 - Folded product: L37 x W52.5 x H89 cm
 - Seat size: W 35.5 x L 30 cm
 - Maximum seat load: 125 kg
 - Maximum load in the removable inner bag: 15/ 36 L kg
 - Net weight of the chassis: 7 kg
- 
- 

ASSIST - INSTRUCTIONS
20

Maintenance

Use a dry cloth for everyday maintenance. If you need to remove any stains, we recommend using a neutral detergent and wiping off the stain manually.

In order to lengthen the life of the product, in the event of rain or contact with water, you should pat the chassis dry with a dry cloth.

Do not modify or dismantle the product because this can lead to malfunctioning, and may even make it no longer safe to use.

EN

Recommendations

This rollator is deemed to meet the Essential requirements and relevant provisions of EC Directive: Medical Devices Directive (EU) 2017/745. The company cannot be held responsible for any other use made of the product.

Do not move the rollator when a person is sitting on the seat.

The maximum weight that the bag holds is 15 kilograms. Do not overload the bag even when the maximum weight has not been reached. Always put the heavier items at the bottom of the bag.

Take special care when using the product on a ramp or a street with a slope, especially if the rollator is loaded. Remember to use the handbrake to control the rollator.

Take special care when using the rollator on wet or slippery ground.

Check that the safety or parking brake is on before using it as a seat.

This product is not a toy. Do not allow children to handle and/or climb on it.

Do not use the rollator to transport pets.

Do not keep the rollator in places that are damp, very hot or exposed to direct sunlight, as this may shorten the life of the product.

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